Feline Leukemia Virus Antigen Test Kit

For veterinary use only.

English version

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The conjugate contains enzyme-conjugated antibody to p27. Upon mixing the conjugate and the test sample, conjugated monoclonal antibody will bind p27 antigen (if present). The sample/conjugate mixture is then added to the SNAP device and flows across the spot matrix. The matrix-bound p27 antibody (FeLV spot) will capture the p27-conjugated antibody complex. The device is then activated, resulting wash and substrate reagents stored within the substrate presents à l'intérieur.

Color development in the FeLV Ag sample spot indicates the presence of FeLV antigen.

Precautions and warnings

- All wastes should be properly decontaminated prior to disposal.
- Do not mix components from kits with different serial numbers.
- Do not use a SNAP device that has been activated prior to the addition of sample.
- Refer to country specific Material Safety Data Sheet for regional hazard identification.

Storage

• SNAP devices and test reagents must be stored at 2–8°C.

Kit components

Item	Reagents	Amount 15 Test	Amount 30 Test
1	bottle Anti-FeLV- HRPO Conjugate (Preserved with gentamicin and ProClin™ 150)	3.5 mL	7.0 mL
2	SNAP Device - Monoclonal antibodies to p27 and positive and negative controls	15	30
Reagents contained in each device:			
Wash Solution (preserved with ProClin™ 150)	0.4 mL	0.4 mL	
Substrate Solution	0.6 mL	0.6 mL	
Other components: Transfer pipettes, sample tubes, and reagent rack			

Conjugé - H317 / H402 / H412 / P261 / P260 / P333+P313: May cause an allergic skin reaction. Harmful to aquatic life. Harmful to aquatic life with long lasting effects. Avoid breathing mist/vapors. Wear protective gloves. If skin irritation or rash occurs: Get medical advice/attention.

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