## Canine Leishmania Antibody Test Kit

For veterinary use only.

English version 

### SNAP® Leishmania

The Canine Leishmania Antibody Test Kit is an Enzyme-linked Immunosorbent assay for in vitro detection of Leishmania donovani or Leishmania infantum (*L. donovani* or *L. infantum*) antibodies in canine whole blood, serum, or plasma.

#### Precautions and warnings

- All waste should be properly decontaminated prior to disposal.
- Do not mix components from kits with different serial numbers.
- Do not use a SNAP device that has been activated prior to the addition of sample.
- Refer to country specific Material Safety Data Sheet for regional hazard identification.

#### Storage

SNAP devices and test reagents must be stored at 2-8°C. All components must be at room temperature (18-25°C) before running the test - **DO NOT HEAT**.

#### Kit components

Item	Reagent	Quantity	Quantity
1	Leishmania Antigen-HRPO Conjugate	1 x 6.0 mL	2 x 6.0 mL
2	SNAP Device	10	30
<b>Reagents contained in each device:</b>			
Wash solution	0.4 mL	0.4 mL	
Substrate solution	0.6 mL	0.6 mL	
Other components: transfer pipette, sample tubes and reagent rack			

**Conjugate / Wash Solution:** H317/H412/P261/P280/P302+P352/P333+P313: May cause an allergic skin reaction. Harmful to aquatic life with long lasting effects. Avoid breathing mist/vapors. Wear protective gloves. IF ON SKIN: Wash with plenty of soap and water. If skin irritation occurs: Get medical advice/attention.

**Substrate Solution:** H317/P280/P302+P352/P333+P313: May cause an allergic skin reaction. Wear protective gloves. IF ON SKIN: Wash with plenty of soap and water. If skin irritation or rash occurs: Get medical advice/attention.


**WARNING:** This product contains a chemical known to the State of California to cause cancer.

**Sample information**


- Samples must be at room temperature (18-25°C) before beginning test procedure.
- Serum or plasma, either fresh, previously frozen or stored at 2-8°C, may be used in this test.
- Serum or plasma may be stored for up to 7 days at 2-8°C. For longer storage, sample should be frozen (-20°C or colder).
- Previously frozen or older samples must be centrifuged before use.
- Hemolyzed or lipemic samples will not affect results.
- EDTA or heparin in plasma will not affect results.
- Whole blood may be used. Whole blood must be anticoagulated (eg. EDTA, heparin) and may be used either fresh or after refrigeration (2-8°C) for up to one week.

#### Test procedure

- Using the pipette that is provided, transfer **2 drops of sample** (whole blood, serum, or plasma) into sample tube.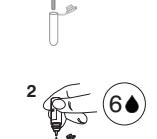


- Holding bottle vertical, add **6 drops** of conjugate to the sample tube.

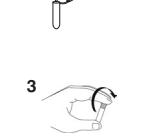
- Cap the sample tube and mix thoroughly by inverting tube **3-5 times**.

- Place the device on a flat surface. Add contents of sample tube to Sample Well, being careful not to splash contents outside of Sample Well. Sample will flow across Result Window, reaching the activation circle in approximately 30-60 seconds. Some sample may remain in Sample Well. Watch carefully for sample or blue color in the activation circle.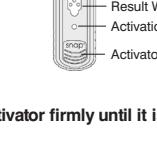

- When color **FIRST** appears in activation circle push activator firmly until it is flush with the device body.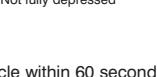


**NOTE:** Some samples may not flow to the activation circle within 60 seconds, and therefore, the circle may not turn color. In this case, press the activator after sample has flowed across Result Window.

• Keep the device horizontal to ensure accurate results.


5. Read result at 6 minutes. **NOTE:** positive control may develop sooner, but results are not complete until 6 minutes.

#### Interpreting test result


To determine test results, read the reactions spots in the Result Window. Color development in Leishmania sample spot is proportional to the Leishmania-specific antibody concentration in the sample. If no color develops in the positive control spot, repeat the test.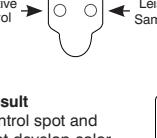

**Negative result**  
Only positive control spot develops color.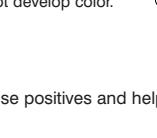

**Reaction with negative control**  
The negative control spot serves as a safeguard against false positives and helps indicate that the assay has been run properly.

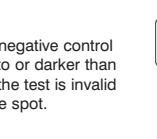**Positive result**  
If color in the sample spot is darker than negative control spot, result is positive for that spot.

**Invalid results**

- Background**  
If the sample is allowed to flow past the Activate Circle, background color may result. Some background color is normal. However, if colored background obscures test result, repeat the test.
- No color development**  
If positive control does not develop color, repeat the test.

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**Symbol descriptions**

	Use by date		Authorized representative in the European Community
	Batch code (lot)		
	Serial number		Consult instructions for use
	Temperature limitation		In vitro diagnostic
	Manufacturer		Date of manufacture
	Catalog number		

#### IDEXX Technical Support

USA/Canada: 1 800 248 2483 • idexx.com

Australia: 1300 44 33 99 • idexx.com.au

Europe: idexx.eu

1. Georgi, JR, George, ME. Canine Clinical Parasitology. Philadelphia, Lea and Febiger, 1992.  
2. El SNAP test gave the same result as the IF in 98% of the samples tested. IF developed by Dr. Trescott (Uv Leo), study performed at Univ of Madrid, 108 IF/40 Positive Samples, 120 IF/40 Negative Samples.

\*SNAP is a trademark or registered trademark of IDEXX Laboratories, Inc. or its affiliates in the United States and/or other countries.

\*\*IDEXX is a trademark or registered trademark of IDEXX Laboratories, Inc. or its affiliates in the United States and/or other countries.

© 2020 IDEXX Laboratories, Inc. All rights reserved. • 06-02749-11

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#### Conjugate / Wash Solution:

H317/H412/P261/P280/P302+P352/P333+P313: May cause an allergic skin reaction. Harmful to aquatic life with long lasting effects. Avoid breathing mist/vapors. Wear protective gloves. IF ON SKIN: Wash with plenty of soap and water. If skin irritation or rash occurs: Get medical advice/attention.

#### Substrate Solution:

H317/P280/P302+P352/P333+P313: May cause an allergic skin reaction. Wear protective gloves. IF ON SKIN: Wash with plenty of soap and water. If skin irritation or rash occurs: Get medical advice/attention.

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IDEXX Europe B.V.